

## **Installing the Pocket Medic™ for HPC Mouthpiece Patch**

1. Launch the QRS website [www.QRSdiagnostic.com](http://www.QRSdiagnostic.com)
2. Browse to Support\Free Software. Login to the page using:  
Username: customer  
Password: obtain from Technical Support
3. Remove all devices from the PC Card Slot(s).
4. Connect the Handheld PC (HPC) to your Personal Computer (PC) via a USB cable or Infrared connection.
5. Run the Windows® ActiveSync® software on your PC to establish a communications link between the PC and the HPC.
6. Select the Pocket Medic for HPC Mouthpiece Patch link on the QRS website.
7. Select Save. Save the file to: "My Handheld PC\Windows\Desktop" directory on the PC.
8. On the HPC desktop, double-click on "MouthpiecePatch3.exe".
9. Select "Turn Off Mouthpiece Lockout".
10. Close the window.