

## **Installing the Pocket Medic™ for PPC Mouthpiece Patch**

1. Launch the QRS website [www.QRSdiagnostic.com](http://www.QRSdiagnostic.com)
2. Browse to Support\Free Software. Login to the page using  
Username: customer  
Password: obtain from Technical Support.
3. Remove all devices from the PC Card Slot(s).
4. Connect the Pocket PC (PPC) to your Personal Computer (PC) via a USB cable or Infrared connection.
5. Run the Windows® ActiveSync® software on your PC to establish a communications link between the PC and the PPC.
6. Select the Pocket Medic for PPC Mouthpiece Patch link on the QRS website.
7. Select Save. Save the file to: "My Windows Mobile Based Device\  
Windows\Start Menu".
8. On the PPC select Programs and then select "MouthpiecePatch2".
9. Select "Turn Off Mouthpiece Lockout".
10. Close the window.